THUASNE

SPORT

fr	Chevillère de maintien Thuasne Sport	3
en	Thuasne Sport ankle support	4
de	Knöchelbandage Thuasne Sport	5
nl	Thuasne Sport steunbrace voor de enkles	7
it	Cavigliera di sostegno Thuasne Sport	8
es	Tobillera de sujeción Thuasne Sport	9
pt	Tornozoleira de suporte Thuasne Sport	11
da	Ankelstøtte Thuasne Sport	12
fi	Thuasne Sport -nilkkatuki	13
sv	Thuasne Sport ankelortos för stöd	14
el	Επιστραγαλίδα συγκράτησης Thuasne Sport	16
cs	Kotníková bandáž Thuasne Sport	17
pl	Podtrzymująca orteza skokowo-stopowa Thuasne Sport	18
lv	Thuasne Sport potītes atbalsts	20
lt	Palaikomasis kulkšnies įtvaras „Thuasne Sport“	21
et	Thuasne Sport hüppeliigese tugi	22
sl	Opornica za gleženj Thuasne Sport	24
sk	Ortéza členku Thuasne Sport	25
uh	Thuasne Sport bokarögzító	26
bg	Наглезеник с опора Thuasne Sport	28
ro	Glezneră de susținere Thuasne Sport	29
ru	Поддерживающий голеностопный ортез Thuasne Sport	30
hr	Potporni steznik za gležanj Thuasne Sport	32
zh	Thuasne Sport 稳定护踝	33
ar	Thuasne Sport دعامة مسك الكاحل	35

 cm	S	M	L	XL	XXL
	19 - 20	21 - 22	23 - 24	25 - 27	28 - 30

fr	Tour de cheville	_____
en	Ankle circumference	_____
de	Knöchelumfang	_____
nl	Omvang van de enkel	_____
it	Giro caviglia	_____
es	Circunferencia del tobillo	_____
pt	Perímetro do tornozelo	_____
da	Anklens omkreds	_____
fi	Nilkanympärys	_____
sv	Runt fotleden	_____
el	Περίμετρος αστραγάλου	_____
cs	Obvod kotníku	_____
pl	Obwód kostki	_____
lv	Potītes apkārtmērs	_____
lt	Kulkšnies apimtis	_____
et	Pahkluu ümbermõõt	_____
sl	Obseg gležnja	_____
sk	Obvod členka	_____
hu	Boka kerülete	_____
bg	Обиколка на глезена	_____
ro	Circumferința gleznei	_____
ru	Окружность голени	_____
hr	Opseg gležnja	_____
zh	脚踝尺寸	_____
ar	محيط الكاحل	_____

fr

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THUASNE SPORT ANKLE SUPPORT

Description/purpose

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - Elastane - Cotton.

Properties/Mode of action

Elastic knit.

Comfort zone over the ankle bones.

Indications

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Mild ankle sprain (acute phase).

Sport activities resumption.

Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,

- In case of changes in the device's performance.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2033201 (2022-02)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186