THUASNE

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SOFT CERVICAL COLLAR

Description/Destination

Rounded edges for appropriate comfort. ❶

Precise self-fastening adjustment. ❷

Convenient removable cover. ❸

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

cotton - polyamide - polyester - polyurethane.

Properties/Mode of action

Support.

Proprioceptive effect.

Indications

Torticollis (stiff neck).

Post-operative use.

Rehabilitation after wearing a rigid cervical collar.

Cervical syndrome.

Dropped head syndrome.

Cervical arthritis.

Mild whiplash/mild sprain.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

Precautions

Strictly comply with your healthcare professional’s prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product’s integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity. In the event of long-term wear

of the product, possible secondary risks related to muscle atrophy. Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Open the self-fastener.

Place the collar around the neck. ❹

Close the self-fastener at the nape of the neck. ❺

Ensure the strap is not fastened too tight. ❻

Care

See product label. Close the self-fastening tabs before washing. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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© Thuasne - 6167501 (2022-02)