THUASNE

Ortel C3




fr	Collier cervical rigide.....	6
en	Rigid cervical collar.....	7
de	Feste Halskrause.....	8
nl	Stijve nekbrace.....	9
it	Collare cervicale rigido.....	11
es	Collarín cervical rígido.....	12
pt	Colar cervical rígido.....	13
da	Stiv nakkekrave.....	14
fi	Jäykkä kaulatuki.....	15
sv	Rigid nackkrage.....	17
el	Αυχενικό κολάρο σκληρό.....	18
cs	Pevný krční límec.....	19
pl	Kołnierz sztywny do unieruchomienia szyi.....	20
lv	Stingra kakla ortoze.....	22
lt	Standus kaklo įtvaras.....	23
et	Jäik emakakaela krae.....	24
sl	Opornica za vrat, toga.....	25
sk	Anatomicky tvarovaná penová opora.....	27
hu	Kemény nyakmerevítő.....	28
bg	Твърда цервикална яка.....	29
ro	Guler cervical rigid.....	31
ru	Жесткий шейный воротник.....	32
hr	Tvrda ortoza za vratnu kralježnicu.....	33
zh	硬质颈圈.....	35
ar	طوق للعنق صلب.....	37

	1	28 - 33 cm
	2	34 - 39 cm
	3	40 - 46 cm

fr	Tour de cou	lv	Kakla apkārtmērs
en	Neck circumference	lt	Kaklo apimtis
de	Halsumfang	et	Reieübermõõt
nl	Omtrek van de nek	sl	Obseg vratu
it	Circonferenza del collo	sk	Obvod krku
es	Circunferencia del cuello	hu	Nyak kerülete
pt	Perímetro do pescoço	bg	Размер на яката
da	Halsens omkreds	ro	Circumferința gâtului
fi	Kaulanpääryn	ru	Обхват шеи
sv	Runt halsen	hr	Opseg vrata
el	Περίμετρος λαιμού	zh	颈围
cs	Obvod krku	ar	محيط العنق
pl	Obwód szyi		

fr	Stabilisation	Immobilisation
en	Stabilisation	Immobilisation
de	Stabilisierung	Ruhigstellung
nl	Stabilisatie	Immobilisatie
it	Stabilizzazione	Immobilizzazione
es	Estabilización	Inmovilización
pt	Estabilização	Imobilização
da	Stabilisering	Støtte
fi	Stabiloi	Immobilisoi
sv	Stabilisering	Immobilisering
el	Σταθεροποίηση	Ακινητοποίηση
cs	Stabilizace	Znehybnění
pl	Stabilizacja	Unieruchomienie
lv	Stabilizēšana	Imobilizācija
lt	Stabilizavimas	Imobilizavimas
et	Stabiliseerimine	Liikumatuks muutmine
sl	Stabiliziranje	Imobilizacija
sk	Stabilizácia	Imobilizácia
hu	Stabilizálás	Rögzítés
bg	Стабилизация	Обезгвужване
ro	Stabilizare	Imobilizare
ru	Стабилизация	Иммобилизация
hr	Stabilizacija	Imobilizacija
zh	稳定	固定
ar	الثبت	التثبيت

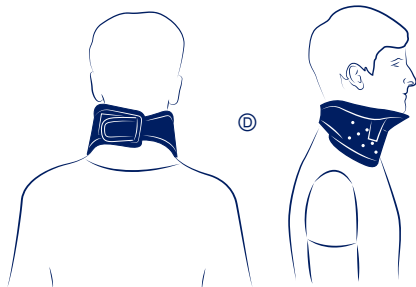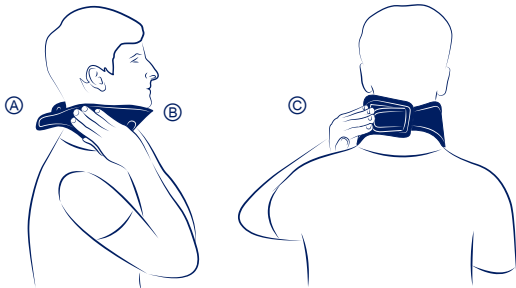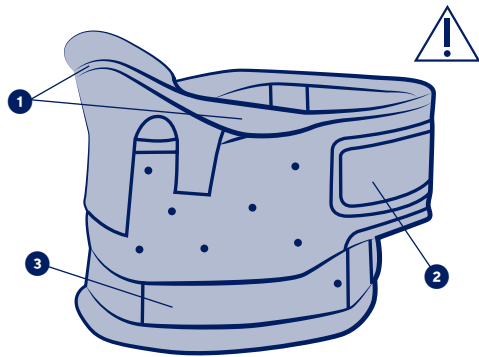MD

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Description/Destination

Anatomically-shaped chin support. ①

Precise self-fastening height adjustment (9 cm to 13 cm). ②

Precise self-fastening adjustment. ③

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Cotton - polyamide - polyester - polyethylene - polyurethane.

Properties/Mode of action

Stabilization of the cervical spine.

Immobilization of the cervical spine in severe injuries.

Indications

Cervical radiculopathy.

Cervical spondylosis.

Stable cervical fractures.

Post-operative use.

Whiplash.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, unstable fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

Do not wear the product when driving a vehicle.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Wearing a cervical collar may cause some weakness of the neck muscles.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Open the self-fastener.

Place the collar around the neck. Ⓐ

The chin should rest on the chin support. Ⓑ

Close the self-fastener at the nape of the neck. Ⓒ

Ensure the strap is not fastened too tight. Ⓓ

Adjust the height if necessary.

Care

See product label. Close the self-fastening tabs before washing. Hand wash.

Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2029101 (2021-03)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros