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en
CLASSIC FOLDING STICK
CLASSIC WALKING STICK WITH A T-SHAPED
HANDLE ART WALKING STICK WITH AN
ANATOMICAL HANDLE

Description/Destination

Folding stick (W2070)/Walking stick (W2090 and W2110/W2111).
This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It can be used inside and outside.

Main components

Tubes: aluminum.
Handle: wood (W2070 and W2090) - ABS (W2110/W2111).
Ferrule: synthetic rubber.

Indications

Post-operative and post-traumatic support.
Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.
Degenerative conditions of the lower limbs.

Contraindications

Not suitable for patients weighing > 100 kg.
Do not use in the event of known allergy to any of the components.

Precautions

Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

It is recommended that a health care professional supervises the first use.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Before each use of this product, check that it is stable and rigid.

In case of any visible damage or significant wear of the ferrule, contact a specialized dealer or health care professional to have it replaced.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles, etc.), uneven or unstable surfaces (gravel, sand, leaves, etc.).

Do not leave exposed to direct sunlight for a long period of time.

Undesirable side-effect

• Soreness, redness, rashes, irritation, blisters, or wounds of varying severity, shoulder joint strain, pain in the hand, wrist, forearm or arm (biceps), hematomas and bruises, etc.

• Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use

Unfolding and folding the Classic Folding Stick W2070:

• Unfolding the walking stick: hold the walking stick vertically with one hand at the tube segments and take the handle in the other hand. Release the tube segments, making sure you point the tip toward the ground. The various segments automatically fit together.

• Folding the walking stick: take the upper segment of the tube and gently pull downward to release and fold. Do the same with the following segments until the walking stick is fully folded.

Indicative height of user:

• Classic Folding Stick W2070: 160 to 180 cm.

• Classic Walking Stick with a T-Shaped Handle W2090: 149 to 196 cm.

• ART Walking Stick with an Anatomical Handle W2110/W2111: 148 to 194 cm.

The height of the walking stick must be adjusted with reference to the height of the patient. Incorrect height adjustment can impair performance and place too much strain on the user.

Walking stick height adjustment:

After adjustment, when standing up straight with one hand on the handle, the user's arm should be bent at an angle of 30°.

The top of the handle must be approximately at the height of the wrist fold when the user's arms are alongside the body, wearing usual shoes (Fig. A).

Adjustment by push-button: unscrew the ring by a quarter of a rotation. Press the push-button while pulling the tube in order to adjust the height. Screw the ring back on without tightening excessively. This enables silent use of the walking stick.

Use:

The walking stick must be placed as vertically as possible on the ground.

The walking stick must be placed between 5 to 10 cm to the side, and approximately 15 cm in front of the foot.

When in use, grasp the handle of the device completely and firmly.

If you fall: let the walking stick slip to the side so that you do not fall on it.

Important warnings

Always keep one hand on the handrail when going up or down stairs.
When it is necessary to take stairs without a handrail, only do so with extreme caution.
Only use the device as intended and not for any other use: pulling, lifting or moving objects.

Maintenance and care

Maintenance

If any part of the device becomes detached or loose, contact a specialized dealer or health care professional.

If necessary, replacement parts and original accessories are available from specialized dealers for the whole range of Thuisne products.

Care/Maintenance

Clean with a mild cleaning agent and a soft cloth.

Do not clean the handle or the ferrule with greasy substances, this could create a slip hazard.

If necessary, use a mild contact disinfectant or 70% alcohol.

Storage

Store at room temperature.

Disposal

Dispose of in accordance with local regulations.

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