**THUASNE**

# Chaise toilettes 3 en 1

W1910 001 001

<b>fr</b> Chaise toilettes 3 en 1 .....	3
<b>en</b> 3 in 1 commode .....	4
<b>de</b> 3-in-1-Toilettenstuhl.....	5
<b>nl</b> 3-in-1 toiletstoel .....	6
<b>it</b> Sedia da comodo 3 in 1.....	7
<b>sk</b> Toaletné kreslo 3 v 1.....	8

fr

## CHAISE TOILETTES 3 EN 1

### Description/Destination

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en  
**3 IN 1 COMMODE**

**Description/purpose**  
 The 3-in-1 commode is designed for people with reduced mobility. It is only for indoor use by adults.  
 The 3 in 1 commode consists of:

1. A variable-height commode with pail, toilet seat and lid for people with reduced mobility who cannot easily reach the bathroom.
2. A toilet seat raiser with anti-splash system, seat and lid enabling the height of the toilet seat to be adjusted for people with difficulty standing.
3. A standard-height toilet frame providing support for standing up.

- Main components**
- Epoxy-treated steel
  - Toilet seat and lid: polypropylene
  - Armrests: polyethylene
  - Connecting sleeves: polyamide and glass fiber
  - Ferrules: PVC

**Indications**  
 Post-operative and post-traumatic support.  
 Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.  
 Degenerative conditions of the lower limbs.

**Contraindications**  
 Not suitable for patients weighing > 110 kg.  
 Do not use in the event of known allergy to any of the components.  
 Do not use in the event of severe perceptual function deficit or severe balance disorders.

**Precautions**  
 **Important warnings**

Do not leave or store the product outside.  
 Do not use in the shower.  
 If you have problems with your balance, ask someone else to help you sit down and stand up again.  
 Follow the advice of the healthcare professional who prescribed or supplied the product and consult them in case of doubt.

It is recommended that a healthcare professional assembles and adjusts the device and supervises the first use.  
 Verify the product's integrity before every use. Do not use the product if it is damaged.  
 Before using this product, check that it is stable and rigid and that all the components are properly attached. Also check the condition of the ferrules.

Do not use the product for any purpose other than its intended use: risk of injury or damage to equipment.  
 Beware of the risks of pinching when handling the device.  
 Monitor the use of the product by people with limited physical capacity or people with cognitive impairments. Never allow children to play with the product.  
 Take care when using the product on slippery surfaces (wet floors, damp tiles etc.).

Do not pick up or move the product while the user is sitting on it: risk of injury or damage to equipment.  
 Do not make any mechanical changes to the product, as this could compromise its safety. Only original spare parts may be used.

- Undesirable side-effects**
- Risk of discomfort in the event of incorrect adjustment of the device.
  - Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

- Instructions for use**  
**Assembly**
1. Unfold the frame.
  2. Straighten the pail support, fastening it to the right and left sides of the frame using the push buttons.
  3. Place the back support bar at the back of the frame.
  4. Adjust the feet to the desired height and make sure they are set to the same height.

 After assembly of points 2, 3 and 4, make sure the push buttons are properly inserted into the holes.  
 Position the pail or the anti-splash system depending on the desired use and clip the toilet seat and its lid in place by pressing firmly on the frame.

 Clean the product thoroughly before first use.

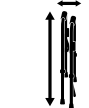
**Maintenance and care**  
**Maintenance**  
 If any part of the product becomes visibly damaged, contact a specialized dealer or healthcare professional.  
 If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

**Care**  
 Clean with a mild cleaning agent and a soft cloth.  
 If necessary, use a mild contact disinfectant or 70° alcohol.  
 In an institution, follow the recommendations of the hygiene manager.

**Storage**  
 Store at room temperature, in a dry, ventilated place.

**Disposal**  
 Dispose of in accordance with local regulations.

**Technical characteristics**

Overall width/depth  57 x 58 cm	Overall height (with lid lowered)  63 - 78 cm	Height of bucket support tubes  38 - 53 cm
Width between armrests  46 cm	Pail width/height  28 x 16 cm	Folded chair height/width  62 x 12 cm
Maximum user weight  110 kg max		Weight  6.25 kg

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©Thuasne - 2034401 (2022-06)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186