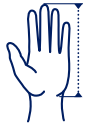
**THUASNE**

# Ligaflex Finger®

<b>fr</b>	Attelle d'immobilisation poignet-doigt .....	5
<b>en</b>	Wrist-finger immobilisation splint .....	5
<b>de</b>	Immobilisierungsschiene für Handgelenk und Finger .....	6
<b>nl</b>	Immobilisatiebrace voor pols en vinger .....	7
<b>it</b>	Tutore di immobilizzazione polso-dita .....	8
<b>es</b>	Férula de inmovilización muñeca-dedo .....	9
<b>pt</b>	Tala de imobilização para o punho/dedo .....	10
<b>da</b>	Skinne til immobilisering af håndled-finger .....	11
<b>fi</b>	Ranteen ja sormien immobilisaatiolasta .....	11
<b>sv</b>	Immobilieringsskena till handled-finger .....	12
<b>el</b>	Νάρθηκας ακινητοποίησης καρπού-δακτύλου .....	13
<b>cs</b>	Imobilizační ortéza zápěstí a prstů .....	14
<b>pl</b>	Orteza unieruchamiająca palec i nadgarstek .....	15
<b>lv</b>	Rokas pirksta imobilizācijas šina .....	16
<b>lt</b>	Riešo ir pirštų imobilizavimo įtvaras .....	17
<b>et</b>	Randme-sõrme fikseerimise lahas .....	18
<b>sl</b>	Opornica za imobilizacijo zapestja in prsta .....	18
<b>sk</b>	Dlaha na znehybnenie zápästia a prstov .....	19
<b>hu</b>	Csukló-ujjrögzítő sín .....	20
<b>bg</b>	Шина за обездвижване китка-пръст .....	21
<b>ro</b>	Atelă de imobilizare încheietura mâinii-deget .....	22
<b>ru</b>	Шина для иммобилизации запястья-пальца .....	23
<b>hr</b>	Udlaga za imobilizaciju ručnog zgloba .....	24
<b>zh</b>	手腕-手指固定夹板 .....	24
<b>ar</b>	جبيرة لتثبيت الرسغ والأصابع .....	27

	1	< 13 cm
	2	13 - 16 cm
	3	16 - 19 cm
	4	19 - 22 cm

<b>fr</b>	Distance bout doigt lésé-poignet - Modèle bilatéral
<b>en</b>	Tip of injured finger – wrist distance (in cm) - Bilateral model
<b>de</b>	Abstand zwischen Fingerspitze des verletzten Fingers und Handgelenk - Beidseitig tragbares Modell
<b>nl</b>	Afstand top gekwetste vinger-pols - Bilateral model
<b>it</b>	Distanza dal polso alla punta del dito lesa - Modello bilaterale
<b>es</b>	Distancia extremo dedo lesionado-muñeca - Modelo bilateral
<b>pt</b>	Distância da ponta do dedo lesionado - punho - Modelo bilateral
<b>da</b>	Afstand skadet finger (spids) - håndled / Størrelse - Bilateral model
<b>fi</b>	Etäisyys vahingoittuneesta sormesta ranteeseen - Molemmipuolinen malli
<b>sv</b>	Avstånd mellan skadad fingertopp - handled - Bilateral modell
<b>el</b>	Απόσταση άκρης τραυματισμένου δακτύλου - καρπού - Αμφίπλευρο μοντέλο
<b>cs</b>	Vzdálenost špičky poškozeného prstu a zápěstí - Bilaterální model
<b>pl</b>	Odległość od zakończenia uszkodzonego palca do nadgarstka - Model dwustronny
<b>lv</b>	Attālums līdz traumētā pirksta galam no plaukstas locītavas – divpusējs modelis
<b>lt</b>	Atstumas nuo riešo iki sužaloto piršto galo – Abiem pusėms tinkantis modelis
<b>et</b>	Kaugus haigest sõrmest randmeni - Kaheosaline toode
<b>sl</b>	Razdalja na poškodovanem prstu med vrhom prsta in zapestjem - Dvostranski model
<b>sk</b>	Vzdialenosť od špičky poraneneho prsta – zápästie Bilaterálny model
<b>hu</b>	A sérült ujjhegy és a csukló közötti távolság - Mindkét oldalon alkalmazható típus.
<b>bg</b>	Разстояние от върха на увредения пръст до китката - Модел, подходящ за двете ръце
<b>ro</b>	Distanță vârf deget vătămat - incheietură la mână - Model bilateral
<b>ru</b>	Расстояние между концом пальца-запястьем - Двусторонняя модель
<b>hr</b>	Udaljenost između ozlijeđenog vrha prsta-ručnog zgloba - Može xse nositi s obje strane
<b>zh</b>	受伤手指指尖与腕部的距离 - 双侧型号
<b>ar</b>	المسافة بين طرف الأصبع المرحوح - الرسغ - طرفان تائي

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## WRIST-FINGER IMMOBILISATION SPLINT

### Description/Destination

This device is a wrist-finger splint providing immobilisation of the fingers.

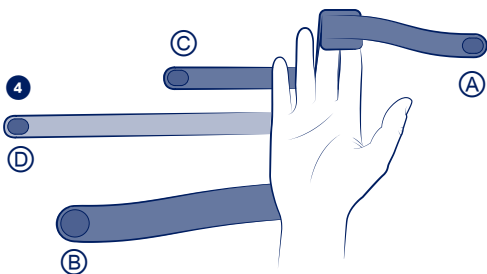
The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Polyamide - elastan - ethylene vinyl acetate - polyethylene - polyester - aluminium.

### Properties/Mode of action

Finger immobilisation splint (2<sup>nd</sup>, 3<sup>rd</sup> or 4<sup>th</sup>).

Conformable and adjustable stay, adapts to the required angulation (flexion or extension).

The splint features 4 straps (2 fixed and 2 removable) to adjust support and tightening. ❶

#### Indications

Finger injuries (sprains, dislocations, ligament ruptures...).

Post-operative/post-traumatic immobilisation.

#### Contraindications

Do not use the product if the diagnosis has not been confirmed.  
Do not use in the event of known allergy to any of the components.  
Do not apply the product in direct contact with broken skin.  
Do not use in case of unstable fractures.

#### Precautions

Verify the product's integrity before every use.  
Do not use the device if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart.  
It is recommended that a healthcare professional supervises the first application.  
Strictly comply with your healthcare professional's prescription and recommendations for use.  
For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

#### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### Instructions for use/Application

According to the complaint and medical advice, the product may be fitted in the palmar position or in the dorsal position.

The injured finger can be immobilised on its own or using double-finger support (injured finger + healthy finger).

Configure the stay according to the type of support required:

- Metacarpophalangeal joints: maximum flexion. ❷

- Interphalangeal joints: extension. ❸

Position the brace, ensuring that the fingertip(s) is/are inserted into the upper end of the brace. ❹

Fasten the straps in the order shown below @ @ @ @. ❺

Tighten the straps reasonably well for proper use of the device (sufficient support).

#### Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label.

Machine washable at 30°C (delicate programme). If possible use a washing net. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

#### Storage

Store at room temperature, preferably in the original packaging.

#### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2027201 (2021-05)